# CHARISMA® Diamond

66044645-100 - 03-10

- Ⓢ** Instructions for use
- Ⓔ** Instruções de uso
- Ⓣ** Instruções de uso
- Ⓐ** تعليمات الاستخدام
- Ⓢ** 使用说明
- Ⓚ** 사용 설명서
- Ⓓ** Petunjuk penggunaan

## CHARISMA® Diamond

**Product description:** CHARISMA Diamond is a light curing, radio-opaque nano composite. It is indicated for adhesively bonded, both colored restorations. It is suitable for anterior and posterior applications. Due to the nano filler system, a beautiful esthetic result and a durable, high lustre polish is guaranteed.

**Composition:** CHARISMA Diamond is based on a TCD-DI-HEA and UDMA (contains 64% filler by volume, 5 mm–20 µm) • Barium Aluminium Fluoride glass • Highly discrete nanoparticles

**Available Colors:**

Opaque Dentin	OB (Opaque Bleach)	BL (Bleach Light)	OL (Opaque Light)	OM (Opaque Medium)	OD (Opaque Dark)
	BXL (Bleach Extra Light)	BL (Bleach Light)	A 1	A 2	A 3
			B 1	A 2	A 3
Universal				C 2	B 3
				D 3	C 3
Incisal	translucent opalescent	CL (Clear), AM (Amber)	CO (Clear Opal), YO (Yellow Opal)		

**Indications:**

- Direct restoration of Class I–V cavities (according to G.V. Black) • Direct composite veneers • Shape corrections of teeth (ie. diastemas, congenital defects in teeth, etc.) • Splinting of teeth loosened by trauma or periodontal disease • Indirect restorations (inlays, veneers)
- Restorations: • Replacement of primary teeth • Core build-up • Repairs of coronal, composite (in combination with an adequate repair-system)

**Contraindications:** Application to dentine close to the pulp (less than 1 mm). Do not use in case of known allergy to any of the constituents of CHARISMA Diamond.

**Side effects:**

- One of its components may in particular cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.
- Further recommendations and precautions: • May cause sensitization by skin contact. • Irritating to eyes. • Avoid contact with the skin and eyes. • In case of contact with skin, immediately wash with plenty of water and soap. • Immediately and thoroughly rinse with plenty of water in case of contact with eyes and contact with ophthalmology. • The P.L.T (post-cured layer) are intended for single use only and cannot be restorable or contaminating the restorative material because the solutions contain phenolic constituents. To avoid cross contamination between patients, discard P.L.T after each single patient use. • In the case of deep cavity preparations in close proximity to the pulp or a direct pulp exposure, cover the dentin or the exposure locally with a calcium hydroxide liner covered with a glass ionomer cement before restoring with CHARISMA Diamond. • Do not use eugenol based restorative resins or an eugenol based primer because these may interfere with the proper curing of CHARISMA Diamond.

**Method of application:** Select an appropriate shade or shade combination from the CHARISMA Diamond Shade guide prior to placing the rubber dam. This is important because teeth will significantly change color when desiccated. If the tooth surface being treated is not to be instrumented by a diamond

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